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ENGLISH

1. Orthodontic Springs

Indication: NITI Springs are indicated as accessories in orthodontic treatment to open or close space, depending on the spring to be used, through the light and continuous mechanical force exerted when engaged to the archwires, brackets or tubes.

2. Purpose: the purpose of the NITI Springs is to exert a light and continuous force that is transferred to the tooth, promoting the movement of the teeth in orthodontic treatment. They are applied to the archwire, brackets and tubes.

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The use of this product is exclusive in the interior of the oral cavity, being applied on the archwires, brackets and tubes bonded to the teeth. This product was exclusively developed for application in the orthodontic treatment and for human use, being restricted to dental professionals with qualification in Orthodontics.

Select the spring to be used, according to the force and tooth movement intended.

Place the spring in the archwire, brackets and tubes bonded to the tooth.

6. Precautions: Product for exclusive use by dental professionals with qualification in Orthodontics.

7. Warnings: It is the orthodontist responsibility to advice patients and legal guardians on the action to be taken in the event of breakage or detachment of the parts and aspiration or swallowing by the patient. It is the professional responsibility the orientation about what should be done in the event of accidents.

It is the primary responsibility of the orthodontist to instruct the patient for proper oral hygiene, to avoid the appearance of plaque or gum disease.

This product contains small amounts of nickel and chromium in its composition and should be avoided in patients with a history of allergy or sensitivity to these metals.

At the slightest sign of sensitization, the patient should be referred to the physician.

This product did not have its biocompatibility and safety evaluated in the MRI (Magnetic Resonance Imaging) environment, the patient should be advised about this condition.

8. Contraindications: Aditek supplies products only to dental professionals with qualification in Orthodontics. It is the primary responsibility of the orthodontist to identify any possible contraindication that may preclude the use of the product, such as patient with poor oral hygiene, existing bone or root resorption, existing

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decalcification of dental enamel, existing periodontal complications, patients with allergy to any of the orthodontic materials used in the treatment, and other conditions that can make treatment difficult, such as patients unable to cooperate with treatment.

9. Side Effects: It is the primary responsibility of the orthodontist to identify any possible risk of injury and/or contraindications that may arise during treatment, relay any possible unwanted side effect to the patient, such as tooth discolorations or decalcification, root resorption, periodontal complications, allergic reactions, difficulties in oral hygiene maintenance, damage to the oral mucosa and tongue, difficulty in speech or chewing, discomfort and pain.

10. Product for single use only: Aditeks' products are designed and manufactured for single use only. Once removed from the patient's mouth, must be disposed of properly.

Considering that the product is used in contact with the mucosa and oral fluids, it is recommended to the professional to apply the sanitary norms applicable for the disposal of medical products, according to current sanitary regulations.

The reuse or reprocessing may cause cross infection and loss of mechanical properties due to natural wear.

11. Product non sterile: The product is supplied in the unsterile condition, however it has proven biocompatibility and no cleaning, disinfection or sterilization processes are required. If the package has been breached, it is recommended to discard the product, minimizing the risk of transmission of microorganisms or possible product adulteration.

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Aditek do Brasil Ltda. is not responsible for the results obtained by using the product without necessary precautions or non-compliance with the warnings and instructions for

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