Scan for more information

Check out our learning platform Reliance U! www.RelianceU.tech

Reliance Orthodontic Products, Inc.
1540 W. Thorndale Avenue,
Itasca, IL 60143 USA
1-800-323-4348 | Made in USA | Rx Only
RelianceOrthodontics.com


EMERGO EUROPE
Prinssgracht 20, 2514 AP The Hague,
The Netherlands

AUSTRALIAN SPONSOR
EMERGO AUSTRALIA
Level 20, Tower II, Darling Park, 201 Sussex St,
Sydney, NSW 2000, Australia

GOTO® FLORESCING LIGHT CURE ORTHODONTIC ADHESIVE

INSTRUCTIONS FOR USE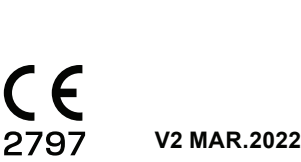

RelianceOrthodontics.com

EN - ENGLISH

INDICATIONS FOR USE: GoTo® is intended for use as a light cure bracket adhesive.

INSTRUCTIONS FOR USE:

Tooth Preparation: Prophyl the enamel surfaces with 1st & Final®, rinse and dry thoroughly.

Etching: Apply etching agent onto the entire area to be bonded. Allow 30 seconds for etching. Rinse and dry thoroughly.

Primer Application: Apply 1 coat of Assure Plus and dry with air.

Application of Paste to bracket: Apply adhesive to bracket pad and smear into the bracket mesh. Apply bracket to the primed tooth surface. Light cure adhesive at close proximity for 10 seconds*. Note: GoTo will fluoresce under the presence of a blacklight

Bracket and Adhesive Removal:

To remove GoTo™ use a ligature cutter to grab the adhesive at the bracket/enamel interface and pinch the bracket from the tooth. Remaining material can be removed with a Renew® Finishing System Bur and Renew® Finishing System #383 Point.

⚠ **PRECAUTIONS:** Unpolymerized resins may cause skin sensation in susceptible persons. In case of contact with the skin, wash thoroughly with soap and water. If accidentally instilled into eye, flush with copious amounts of water and seek medical attention immediately. Contact with other tissues: rinse spill area immediately with copious amounts of water for several minutes. Avoid placement of etching agent on soft tissue.

⚠ Use of UV, halogen and LED curing lights can be associated with eye damage. The patient, doctor and the assistant should wear properly designed protective eyewear when using these light curing devices.

⚠^{EN} **TEMPERATURE LIMITATION:** Store at room temperature (15-30 Degrees C.)

WARRANTY: Reliance Orthodontic Products, Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontic Products, Inc. does not accept liability for any damages or loss, direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connections therewith.

RX ONLY: U.S. Federal law restricts this device to sale by or on the order of a dental professional.

NOTICE: Report any serious incident to Reliance and the competent authority of the state in which the user and/or patient is established.

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