Scan for more information

Check out our learning platform Reliance U! www.RelianceU.tech

Reliance Orthodontic Products, Inc.

1540 W. Thorndale Avenue, Itasca, IL 60143 USA

1-800-323-4348 | Made in USA | Rx Only
RelianceOrthodontics.com


EMERGO EUROPE
Prinsessgracht 20, 2514 AP The Hague, The Netherlands

AUSTRALIAN SPONSOR EMERGO AUSTRALIA
Level 20, Tower II, Darling Park, 201 Sussex St, Sydney, NSW 2000, Australia

RELIANCE GEL ETCHANT BLUE GEL ETCHANT WITH BAC INSTRUCTIONS FOR USE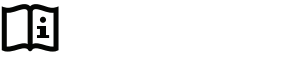

RelianceOrthodontics.com

EN - ENGLISH

INDICATIONS FOR USE: Reliance Gel Etchant is 37% phosphoric acid (H3PO4) polymer-thickened and intended for use on enamel to prepare the surface for bonding.

INSTRUCTIONS FOR USE

- Unscrew and remove the white storage cap from the tip of the syringe.
- Snap the needle tip dispenser in place and twist clockwise until needle is locked.
- Isolate the teeth for etching. Apply etching agent onto the entire area to be bonded. Allow 30 seconds for etching.
 - CAUTION: Avoid placing etching agent on soft tissue.
- Rinse thoroughly and air dry
- When procedure is completed, remove needle tip, discard needle tip and replace the white storage cap.

⚠ **CAUTION:** Etching gels (contain phosphoric acid) are severe eye and tissue irritants. If accidentally instilled into eye, flush with copious amounts of water and seek medical attention immediately. Contact with other tissues: rinse spill area immediately with copious amounts of water for several minutes. Injury may result if etchant is allowed to remain on the skin or mucosa for extended periods of time.

CAUTION: DO NOT REUSE IF SYRINGE BECOMES CONTAMINATED. AVOID CONTACT OF THE REUSABLE PARTS WITH THE PATIENTS MOUTH. USE NEW, UNCONTAMINATED GLOVES. APPLY DISPOSABLE BARRIER SLEEVES/WRAPS BEFORE EACH PATIENT. TIPS ARE SINGLE USE DEVICES. A NEW TIP IS REQUIRED FOR EACH PATIENT.

🌡 **TEMPERATURE LIMITATION:** Store at room temperature (15-30°C)

TO REORDER: #EGSS, #BEGS, #19TIPS10 **WARRANTY:** Reliance Orthodontic Products, Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontic Products, Inc. does not accept liability for any damages or loss, direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connections therewith.

RX ONLY: U.S. Federal law restricts this device to sale by or on the order of a dental professional. **NOTICE:** Report any serious incident to Reliance and the competent authority of the state in which the user and/or patient is established.

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