## Scan for more information

## Check out our new learning platform Reliance U! www.RelianceU.tech

### Reliance Orthodontic Products, Inc.

1540 W. Thorndale Avenue, Itasca, IL 60143 USA

1-800-323-4348 | Made in USA | Rx Only  
RelianceOrthodontics.com

### EMERGO EUROPE

Prinsessgracht 20, 2514 AP The Hague, The Netherlands

### AUSTRALIAN SPONSOR

#### EMERGO AUSTRALIA

Level 20, Tower II, Darling Park, 201 Sussex St, Sydney, NSW 2000, Australia

## RENEW® FINISHING SYSTEM POINTS

### INSTRUCTIONS FOR USE

  
2797

V1 NOV.2021

RelianceOrthodontics.com

### ENGLISH TRANSLATION

**INDICATIONS FOR USE:** The Renew® Finishing System Points are intended for use in the removal of thin layers of composite and polishing of enamel or artificial substrates.

#### INSTRUCTION FOR USE:

1. Ensure that the point is fully seated and gripped in the handpiece collet before use.

**2. IMPORTANT! Do not exceed the maximum speed of 20,000 RPM.** Find the setting for your handpiece that will ensure a speed not to exceed 20,000 RPM and adjust your air pressure accordingly.

#### SAFETY IN USE:

- Eye protection must be worn to protect against ejected particles.
- Surgical mask must be worn to avoid inhalation of aerosol or dust generated.
- Move the point continuously when in use to avoid localized heating.

Ⓣ Single Use Device

**⚠ WARNING! Used polishing points should be considered as contaminated and appropriate handling precautions should be taken. Gloves, eye protection and a mask should be worn. Other measures may be required if there are specific infection or cross-contamination risks from the patient.**

**LIMITATIONS OF PROCESSING:** The end of life of the polishing points are determined by wear and damage in use and the points should be inspected for defects during the preliminary cleaning process.

**WARRANTY:** Reliance Orthodontic Products, Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontic Products, Inc. does not accept liability for any damages or loss, direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith.

**RX ONLY:** U.S. Federal law restricts this device to sale by or on the order of a dental professional.

**REORDER INFORMATION:** Friction Grip (REF# RSPFG), Handpiece (REF# RSPHP), Latch (REF# RSPL)

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