

Maximum Cure®

CHEMICAL CURE SEALANT SYSTEM WITH FLUORIDE

INDICATIONS FOR USE: Maximum Cure® Sealants are intended for use in direct and indirect bonding of brackets and appliances.

INSTRUCTION FOR USE:

1. Thoroughly prophylax the tooth surface with a paste which does not contain oil. Rinse and dry.
2. Acid etch each tooth for 30 seconds.
3. Rinse the acid from each tooth for 5 seconds and dry with oil-free air.
NOTE: If the enamel is atypical, Assure® Universal Bonding Resin can be applied at this point.
4. On a mixing pad, place 1 drop each of Maximum Cure® Sealant A & B. Do not dispense Sealant A or B until ready to mix and apply.
5. Immediately mix with a brush or sponge pellet.
7. Apply the mixed sealant to the etched enamel surface. The aforementioned mix ratio will produce a 30 second to 1.5 minute working time from the start of mix.
8. Proceed with the application of any two paste or light cure bonding paste to each bracket or appliance. Brackets and appliances can be placed before the sealant totally sets without interference to the sealant or bonding paste. If utilizing a light cure paste, wait 3 minutes prior to seating a bracket.

INSTRUCTIONS FOR USE WITH CUSTOM BASE INDIRECT BONDING PROCEDURE

1. After preparing the enamel as described above, the operator has 2 choices for use as an indirect adhesive:
 - A.) Mix Maximum Cure part A & B and apply a thin coat to the enamel. Then, apply a slightly thicker coat to each custom pad and immediately seat tray.
OR
 - B.) Apply a moderate coat of Maximum Cure part A to the custom pad and part B to the enamel. Proceed to seat the tray.

In either case, hold the tray for 90 seconds and wait 5 minutes before removing tray from mouth.

Special Safety Notice: Brush applicators are single use devices. It is required that a new brush applicator is used for each application to prevent contamination.



PRECAUTIONS: Unpolymerized resins may cause skin sensation in susceptible persons. In case of contact with the skin, wash thoroughly with soap and water. If accidentally instilled into eye, flush with copious amounts of water and seek medical attention immediately. Contact with other tissues: rinse spill area immediately with copious amounts of water for several minutes.



Flammable



Hazardous to aquatic life



TEMPERATURE LIMITATION: Store refrigerated (2-8 Degrees C) Avoid excessive heat.

TO REORDER: Regular: Ref# MC7AF 7g Part A, Ref# MC7B 7g Part B, Ref# MC20AF 20g Part A, Ref# MC20B 20g Part B

Filled: Ref# MCF7AF 7g Filled Part A, Ref# MCFSB 7g Filled Part B,

WARRANTY: Reliance Orthodontics Products, Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontics Products, Inc. does not accept liability for any damages or loss, either direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith.

Rx Only: U.S. Federal law restricts this device to sale by or on the order of a dental professional.

Check out our new learning platform Reliance U!
www.RelianceU.tech



Scan for more information



Reliance Orthodontic Products, Inc.

1540 West Thorndale Ave., Itasca, IL 60143 U.S.A.
1-800-323-4348 · www.RelianceOrthodontics.com



EC

REP

EMERGO EUROPE
Prinsessegracht 20
2514 AP The Hague
The Netherlands

Australian Sponsor: Emergo Australia
201 Sussex St. Darling Park
Tower II, Level 20,
Sydney, NSW 2000 Australia

Made in the USA