SELECIONE O IDIOMA

SELECT THE LANGUAGE

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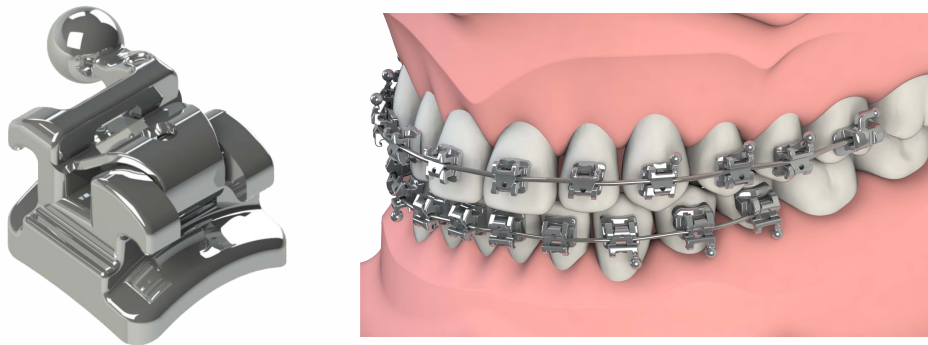
### Self Ligating Brackets

Self Ligating brackets are devices installed by clinical procedure by an orthodontist, bonded to the buccal surface of the tooth, aiming to:

- Transmit efforts to the dental elements correcting its positioning during the dental treatment;
- Disregard the use of bandages or ligatures wires;
- Self Ligating Brackets have a NiTi Clip that holds the wire inside the slot during the treatment.

#### Use Indication:

Self Ligating Brackets are indicated to assist in the treatment of malocclusions in patients requiring movement of the dental elements. In SLI (Self Ligating Interactive) bracket treatments, the clip pressures on the most gauge wires (from the wire .016"x.022") and allows freedom in the less gauge wires, and in the treatments with SLP (Self Ligating Passive) brackets the clip does not exert pressure on the wire.

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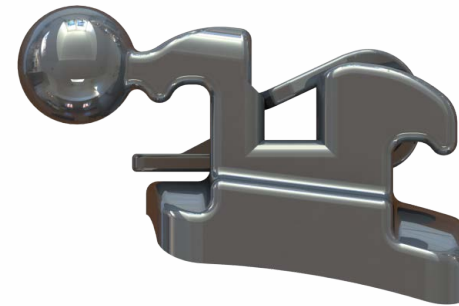
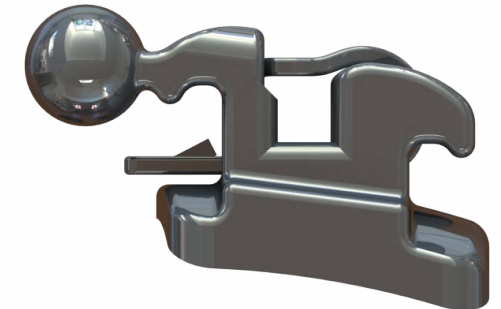
Bracket SLP

#### Bonding:

We recommend using "Orthoprimer" (ref. 85.01.016) and "Orthobond Plus" (ref. 85.01.017).

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All Morelli brackets are sandblasted and undergo a thorough decontamination of the bases.

Do not touch the bonding surfaces before applying adhesive, as any contact can cause contamination, causing adhesive interface failures.

Do not apply disinfection and sterilization methods to the base as they may contaminate or impair the silanization of the ceramic brackets, causing the restraint system to fail.

Do not sandblast, clean or apply other types of silane at the base of any Morelli bracket, as the ideal surface treatment is already done during fabrication.

In case of base contamination, discard and replace the part.

In bonding always use orthodontic adhesives, which promote better adhesion and facilitate the removal of the residual adhesive from the tooth, preserving its enamel. Non-orthodontic resins are denser, penetrate less at the base of the brackets and are difficult to remove and consume more drills.

Proceed with adequate insulation at the time of bonding, avoiding contamination by blood or saliva.

Avoid bonding the bracket with occlusion interference to prevent wear on the tooth surface or premature debonding.

**Debonding of Brackets:**

Use brackets removal instrument.

**Debonding warning:**

Orient the patient NOT TO CHEW OR BITE HARD FOOD during orthodontic treatment to prevent premature detachment.

**Contraindications:**

Morelli supplies the products only to qualified professionals. It is the responsibility of the orthodontist to identify conditions that may be contraindicated for treatment, such as:

- Patients with poor oral hygiene;
- Patients incapable of cooperating with the treatment;
- Patients with propitious oral environment to dental enamel demineralization;
- Allergy to any of the orthodontic device materials;
- Diseases or other pre-existent conditions which may impair orthodontic treatment;
- Root reabsorption;
- Existing bone reabsorption;
- Existing decalcification in dental enamel.

**Side effects:**

It is the responsibility of the qualified professional to identify any possibility of risk of harm and / or contraindication that may arise during treatment, communicate to the patient the possibility of any unwanted side effect.

During treatment, unwanted side effects may include: damage to oral mucosa and tongue, difficulty in speaking or chewing, tooth discoloration, decalcification, root resorption, periodontal complications, allergic reactions, difficulties in maintaining oral hygiene, discomfort, pain, discomfort and sensibility.

**Composition/ Information of components:**

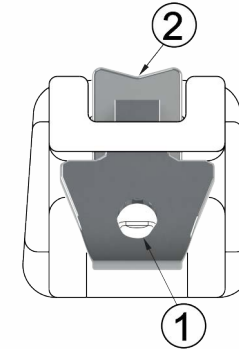
Bracket: Stainless Steel (Cr = 15% - 17,5% / Ni = 3,0% - 5,0% / Cu = 3% - 5% / Mn = Máx 1,5% / Si = Máx 0,7% / Nb = 0,15% - 0,45%)

Clip: Superelastic Nitinol

**Expiry date:**

10 years.

**Warnings:**


**SLP Recommendations**

Open the clip with an explorer tip or similar using the hole (1), and close with your finger. If there is resistance when opening, remove the tartar and use the notch in "V" (2). Close the clip with your finger, making sure to completely cover the slot, as shown.

**SLI Recommendations**

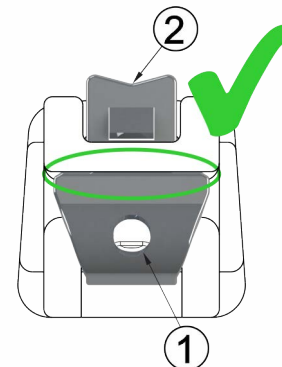In the application of the SLI Brackets it becomes active from the wire .016"x .022". Open the explorer clip or similar tip using the hole (1), and close with your finger. If there is resistance when opening, remove the tartar and use the notch in "V" (2).

When bonding, remove the excess of adhesive to not to clog the clip.

This product hasn't been evaluated for compatibility and safety in Magnetic Resonance environment, the patient should be advised of this condition.

This product contains Nickel and Chromium. A small percentage of the population is allergic to these metals. In case of allergic reactions, advise the patient to seek medical attention.

Non reusable product, as its reuse may cause cross infection and loss of mechanical properties due to natural wear.

The product is supplied in unsterile condition, however has proven biocompatibility, not being necessary cleaning, disinfection or sterilization measures, since the package has not been violated. If the product comes into contact with surfaces or substances other than its intended purpose prior to use, it is recommended to discard the product. The product should not be reused or reprocessed.

It is up to the qualified professional to guide their patients from the risk of aspiration and deglutition in the hypothesis of breakage / detachment of the pieces, should not be underestimated the possible immediate or late complications resulting from this type of accident and must adequately inform the patient and / or their legal guardians about the risks inherent in the event and what actions to take.

Do not use the product if its packaging has been damaged or opened, in order to minimize possible dangers related to the transmission of microorganisms or possible falsification of products.

Product for exclusive application by a qualified professional, the use by person without the necessary technical knowledge may cause unwanted tooth movement, bone loss, loss of the dental element, fenestration, gingival recession or root dehiscence.

It is up to the qualified professional to guide the patient about correct oral hygiene, to avoid the appearance of plaque and tartar, or diseases such as gingival inflammation (gingivitis), periodontitis or even endocarditis.

Considering that the product is used in contact with mucous membranes and body fluids, it is recommended to the professional to apply the sanitary norms applicable for the disposal of medical products according to current sanitary regulations.

#### ATTENTION:

Morelli is not responsible for the results obtained by applying the product without due precautions or non-observance of the warnings.

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