Instructions For Use Enclosed

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Para abrir

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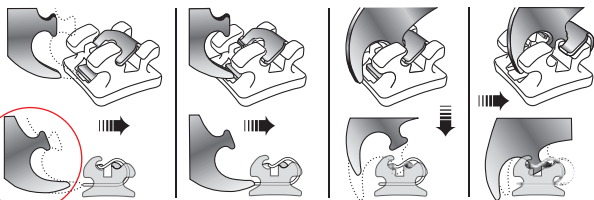
Sensation™
Active Ceramic Self-Ligating
Bracket System

Opening Method — Sensation SLB Opening Instrument # OT-7920

Sensation™ Active Ceramic Self-Ligating Bracket System

Instructions For Use

Opening

To open the Sensation Active Ceramic Self-Ligating Bracket spring clip, place instrument on the open clip slot directly beneath the bracket interface and gently push in the occlusal direction to disengage the bracket clip. Then place the instrument directly in the facial opening of the clip to engage the small extension of the spring clip. Pull in the occlusal direction. The spring clip will pop open.

Closing

To close the Sensation Active Ceramic Self-Ligating Bracket spring clip, simply put light finger pressure on the occlusal surface of the bracket. Make sure the spring clip fully closes so the archwire is securely engaged.

Warning

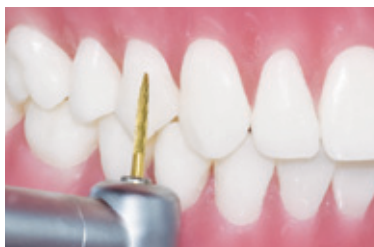
If the bracket and/or clip should become damaged or broken during use, it is recommended to replace the bracket. Follow the debonding procedure to remove the bracket and bonding procedure to replace the bracket as indicated.

Bonding Instructions

For ease of placement and more accurate bracket placement and positioning, close the spring clip on the bracket prior to bonding. It is recommended that bicuspid brackets be placed first, then work to the mesial. This bonding technique will establish the cusp heights in the bicuspid area initially, making it much easier to properly position the remaining brackets and therefore reduce the need to reposition brackets in the future. Next place the canines to correspond to the bicuspid bracket position. Once that is completed, the anterior teeth can be bonded and proper anterior alignment will be more easily achieved. Finally the molar brackets are positioned again using the bicuspid bracket heights as a point of reference.

Debonding Instructions

1. Remove all ligatures and wires.
2. Remove flash from around the brackets with a high speed debonding bur.
3. Instruct patient to bite down onto a cotton roll in the area brackets are being removed. Grasp the bracket with a straight or angled debonding plier at the bracket-adhesive interface. The bracket can be held along the occlusal-gingival or the mesial-distal edge. Gently squeeze the plier and gradually increase pressure until the bracket comes free from the tooth. **Do not twist or pull the bracket.**
4. The bracket is designed to release at the bracket adhesive interface and will leave a layer of adhesive on the enamel for safety. This means that most of the adhesive will remain on the tooth after debonding.
5. Remove remaining adhesive with a high speed debonding bur or an adhesive removing plier.
6. After bulk adhesive is removed follow up with a high or low speed enamel polishing bur.

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