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## ENGLISH

### 1. Self-ligating Brackets

Self-ligating Brackets are devices that, when installed in the dental arches, serve as a support element to transfer the force exerted by the wires and accessories to the teeth. Due to this transfer of forces, the position of the teeth can be changed.

Based on this principle the orthodontic treatment is performed, moving the teeth to the correct and ideal position for a good occlusion.

Self-ligating Brackets have a clip that holds the wire inside the slot, eliminating the use of ligatures or tie wires.

In Passive Self-Ligating Brackets, the clip does not interact actively with the archwire, making the tooth movement gentler and continuous. In Interactive Self-ligating Brackets, the clip interacts with the archwire, by pressing the wire towards the bottom of the slot and increasing torque control.

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#### Bracket – Stainless Steel

Stainless Steel AISI 17-4 – Bracket								
Limits of Chemical elements (%)								
C	Mn	P	S	Si	Cr	Ni	Cu	Fe
≤ 0,070	≤ 1,00	≤ 0,040	≤ 0,030	≤ 1,00	15,0 a 17,5	3,0 a 5,0	3,0 a 5,0	Balance
ASTM: A564								
Name according to EN 10088-3: X5CrNiCuNb 16-4 (1.4542)								

#### Clip – NiTi

Níquel Titânio NiTi #1									
Limits of Chemical elements (%)									
Ni	C	Co	Cu	H	Fe	Nb	N + O	Cr	Ti
54,5 a 57,0	≤ 0,050	≤ 0,050	≤ 0,010	≤ 0,005	≤ 0,050	≤ 0,025	≤ 0,050	≤ 0,010	Balance
ASTM: F2063									

### 3. Shelf life: 10 years

### 4. Instructions for use:

The use of orthodontic Brackets is restricted to dental professionals with qualification in orthodontics, varying according to the technique and prescription selected. You should follow the following procedures:

#### 4.1. Bonding:

It can be used direct or indirect bonding.

Follow the orthodontic adhesive manufacturer's recommendations.

#### 4.2. Archwire insertion:

Open the clip with an explorer, without using force.

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**Easy Opening**

Insert the instrument (explorer) in the circular opening of the clip located in between the lower tie-wings perpendicular to the vestibular face of the tooth.

Gently rotate the instrument to the left or to the right; in case of brackets with hooks, rotate the instrument in the opposite direction of the hook.

The clip may also be open using a fine tip explorer with a gentle lever action downward.

Engage the selected archwire.

Close the clip with a tweezer.

**Secure Closing:**

Place one tip of the tweezer over the upper tie-wings and the other one below the clip.

Gently apply pressure using a tweezer and the clip will begin to move up.

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The clip will click once it disengages from the bottom lock and click a second time when closed.

#### 4.3 Debonding:

Use the appropriate bracket removal instrument.

**5. Precautions:** Product for exclusive use by dental professionals with qualification in Orthodontics.

**6. Warnings:** This product contains small amounts of nickel and chromium in its composition and should be avoided in patients with a history of allergy or sensitivity to these metals. At the slightest sign of sensitization, the patient should be referred to the physician.

This product did not have its biocompatibility and safety evaluated in the MRI (Magnetic Resonance Imaging) environment, the patient should be advised about this condition.

It is the orthodontist responsibility to advice patients and legal guardians on the action to be taken in the event of breakage or detachment of the parts and aspiration or swallowing by the patient. It is the professional responsibility the orientation about what should be done in the event of accidents.

The qualified professional should advise the patient on the correct oral hygiene, in order to avoid the installation of bacterial plaque and consequent periodontal diseases.

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**7. Contraindications:** Aditek supplies products only to dental professionals with qualification in Orthodontics. It is the primary responsibility of the orthodontist to identify any possible contraindication that may preclude the use of the product, such as patient with poor oral hygiene, existing bone or root resorption, existing decalcification of dental enamel, existing periodontal complications, patients with allergy to any of the orthodontic materials used in the treatment, and other conditions that can make treatment difficult, such as patients unable to cooperate with treatment.

**8. Side Effects:** It is the primary responsibility of the orthodontist to identify any possible risk of injury and/or contraindications that may arise during treatment, relay any possible unwanted side effect to the patient, such as tooth discolorations or decalcification, root resorption, periodontal complications, allergic reactions, difficulties in oral hygiene maintenance, damage to the oral mucosa and tongue, difficulty in speech or chewing, discomfort and pain.

**9. Product for single use only:** Aditek's products are designed and manufactured for single use only. Once removed from the patient's mouth, must be disposed of properly.

Considering that the product is used in contact with the mucosa and oral fluids, it is recommended to the professional to apply the sanitary norms applicable for the disposal of medical products, according to current sanitary regulations.

The reuse or reprocessing may cause cross infection, decreased efficiency in bonding and loss of mechanical properties due to natural wear.

**10. Product non sterile:** The product is supplied in the unsterile condition, however it has proven biocompatibility and no cleaning, disinfection or sterilization processes are required. If the package has been breached, it is recommended to discard the

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product, minimizing the risk of transmission of microorganisms or possible product adulteration.

**ATTENTION**

Aditek do Brasil Ltda. is not responsible for the results obtained by using the product without necessary precautions or non-compliance with the warnings and instructions for use.

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