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## ENGLISH

### 1. Nitinol Reverse Curve Archwire

**Indication:** Nitinol Reverse Curve Archwires are indicated for tooth movement in orthodontic treatment, through the light and continuous mechanical intrusion or extrusion forces exerted when engaged to the brackets bonded to the teeth.

Due to its curvature in the horizontal plane, it is indicated to accentuate or to reverse the curvature of the dental arch. Because it is made of an alloy with shape memory property, the force exerted is gradual and continuous, offering greater comfort to the patient.

**2. Purpose:** the purpose of the archwires is to transmit a light and continuous force generated by the interaction between the archwire and the brackets, promoting the movement of the teeth in orthodontic treatment. They are applied to brackets and tubes.

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**4. Shelf life:** 10 years

**5. Instructions for use:**

The use of orthodontic Archwires is restricted to dental professionals with qualification in orthodontics.

Select the archwire to be used, according to the force e tooth movement intended.

Engage the archwire to the brackets and tubes bonded to the teeth.

Tie the archwire with stainless steel ligature tie or elastomeric, or close the clip of the self-ligating bracket.

Cut the excess of wire distal to the buccal tube with a distal end cutter plier so as to avoid poking the soft tissues of the patient.

**6. Precautions:** Product for exclusive use by dental professionals with qualification in Orthodontics.

**7. Warnings:** This archwire has shape memory property, should not receive excessive stresses and bends, which might fracture the archwire.

It is the orthodontist responsibility to advice patients and legal guardians on the action to be taken in the event of breakage or detachment of the parts and aspiration

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or swallowing by the patient. It is the professional responsibility the orientation about what should be done in the event of accidents.

It is the primary responsibility of the orthodontist to instruct the patient for proper oral hygiene, to avoid the appearance of plaque or gum disease.

This product contains small amounts of nickel and chromium in its composition and should be avoided in patients with a history of allergy or sensitivity to these metals. At the slightest sign of sensitization, the patient should be referred to the physician.

This product did not have its biocompatibility and safety evaluated in the MRI (Magnetic Resonance Imaging) environment, the patient should be advised about this condition.

**8. Contraindications:** Aditek supplies products only to dental professionals with qualification in Orthodontics. It is the primary responsibility of the orthodontist to identify any possible contraindication that may preclude the use of the product, such as patient with poor oral hygiene, existing bone or root resorption, existing decalcification of dental enamel, existing periodontal complications, patients with allergy to any of the orthodontic materials used in the treatment, and other conditions that can make treatment difficult, such as patients unable to cooperate with treatment.

**9. Side Effects:** It is the primary responsibility of the orthodontist to identify any possible risk of injury and/or contraindications that may arise during treatment, relay any possible unwanted side effect to the patient, such as tooth discolorations or decalcification, root resorption, periodontal complications, allergic reactions, difficulties in oral hygiene maintenance, damage to the oral mucosa and tongue, difficulty in speech or chewing, discomfort and pain.

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**10. Product for single use only :** Aditeks' products are designed and manufactured for single use only. Once removed from the patient's mouth, must be disposed of properly.

Considering that the product is used in contact with the mucosa and oral fluids, it is recommended to the professional to apply the sanitary norms applicable for the disposal of medical products, according to current sanitary regulations.

The reuse or reprocessing may cause cross infection and loss of mechanical properties due to natural wear.

**11. Product non sterile:** The product is supplied in the unsterile condition, however it has proven biocompatibility and no cleaning, disinfection or sterilization processes are required. If the package has been breached, it is recommended to discard the product, minimizing the risk of transmission of microorganisms or possible product adulteration.

### **ATTENTION**

Aditek do Brasil Ltda. is not responsible for the results obtained by using the product without necessary precautions or non-compliance with the warnings and instructions for use.

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