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IPR-DistanceControl Set

Product description

- Intensiv IPR (InterProximalReduction)-DistanceControl is a manual metal instrument
- 8 metallic elements of different thicknesses with ergonomic grip:
 - 0.10mm
 - 0.15mm
 - 0.20mm
 - 0.25mm
 - 0.30mm
 - 0.40mm
 - 0.50mm
 - 1.00mm
- Height of measuring part: 4mm
- Length of measuring part: 40mm
- Total length of instrument: 50mm
- Plastic clip as fastening system of the 8 elements in the grip
- Sterilizable (with exception of plastic clip) and reusable

Indications

Measurement and control of the teeth distance in the various stages of the IPR technique in orthodontics.

Practical Tips

- IPR-DC is inserted in the vestibular/buccal direction.
- If the designated instrument does not enter into the interdental space created, proceed to measure the interdental space with the thickness gauge immediately below or through a combination of 2 or more instruments.
- If the designated instrument has too much play in the interdental space created, proceed to measure the interdental space with the thickness gauge immediately above or through a combination of 2 or more instruments.
- In order to assemble the Intensive IPR-DistanceControl Set, insert the plastic rivet inside the holes of the 8 superimposed metal instruments and apply pressure to engage the head in the rivet.
- In order to disassemble the Intensive IPR-DistanceControl Set, remove the head from the rivet by applying outward pressure.

Maintenance and sterilisation

- Intensive IPR-DistanceControl is supplied non-sterile. Sterilize the instruments before first use. After each use on patients, the instruments must be disinfected, cleaned and sterilized as well.
- Sterilization should be performed on the instruments without plastic clip.
- Use a single-pulsed or fractionated vacuum autoclave and subvacuum drying. Chemiclave sterilizers may also be used.
- Sterilization must be carried out according to validated procedures.

Risk Warnings

- During insertion pay attention to the soft tissue to avoid injuries.
- Plastic clip disinfected with disinfectant wipes, do not autoclave.

Packaging

Kit containing 8 elements, Ref. IPR-DC Set

Pack containing 3 instruments of the same thickness
Ref. IPR-DC010/3; IPR-DC015/3; IPR-DC020/3; IPR-DC025/3;
IPR-DC030/3; IPR-DC040/3; IPR-DC050/3; IPR-DC100/3

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