DE

Produktbeschreibung

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GB

Product description

- Metal strip, diamond-coated on one sided only with two working parts of different grit sizes: (80µm, blue, 40µm, red)
- One sided, 3 rows of perforation, diamond-coated metal strip
- Perforated anatomical ends for a better gripping
- Height: 2.5mm
- Total length: 80mm
- Thickness: 0.05mm
- Sterilizable and reusable

Indications

- Shaping (80µm) and finishing (40µm) of special excess in proximal area all aesthetic restorations.
- Correction through an efficient and rapid contouring of all existing aesthetic restorations

Practical Tips

- The total length of the strip (80mm) allows an optimal finger support on the adjacent teeth during work.
- To get past the contact area, use the uncoated part of the strip and exert a slight horizontal movement. Do not twist, bend or pull strip too tightly during use.
- After introduction, work with the less abrasive side (40µm, red), in order to facilitate the access of the more abrasive side (80µm, blue). Then begin the finishing procedure using a back-and-forward movement.
- Then apply the 80µm diamond-coated side, with cautious horizontal movements, to model the proximal contours. Next, switch again to the less abrasive side (40µm, red) to complete the procedure with the finishing.
- ProxoContour Coarse must be removed with a vertical movement after repositioning it to the uncoated segment.
- In the event the contact area is very tight, it is useful to start by inserting a wooden wedge to open up a slight separation between the teeth before placing ProxoContour Coarse in the area to be treated.
- After finishing, polish proximal surfaces. We recommend Intensiv ProxoPolish with 15µm and 8µm diamond grit.

Maintenance and sterilisation

- ProxoContour Coarse is supplied non-sterile. Prior to first use on the patient and immediately after each use, the instruments must be disinfected, cleaned, and sterilized.
- Keep ProxoContour Coarse separate from non-stainless steel instruments, such as rubber polishers and abrasive stones.
- If disinfection by immersion is being used, it is important to use only cleaning/disinfection solutions that provide corrosion protection and strictly observe the concentrations recommended by the manufacturer.
- Sterilization must be performed with validated procedures. If possible, use a single-pulsed or fractionated vacuum autoclave and subvacuum drying. Chemiclave sterilizers may also be used.

Risk Warnings

- Tilting or rotating the ProxoContour Coarse metal strip during use can lead to fracture and/or fissure.
- Pulling too forcefully in tight interdental spaces or beginning with the more abrasive (80µm) grit can cause the fissures.
- Bending the ProxoContour Coarse metal strip causes possible breaking points and can lead to detachment of the diamond coating.
- Leaving proximal surfaces roughened with 40µm grain size can lead to undesirable mechanical retention and formation of plaque.

Intensiv ProxoContour Coarse (grit size 80µm and 40µm)
Ref. PXCC8040/6 Package of 6 Strips

Additional models

Intensiv ProxoContour (grit size 60µm and 40µm)
Ref. PXC6040/6 Package of 6 Strips

Intensiv ProxoPolish (grit size 15µm and 8µm)
Ref. PXP01508/6 Package of 6 Strips

Intensiv ProxoStrip (grit size 40µm and 15µm)
Ref. PX4015/6 Package of 6 Strips
Ref. PX4015/12 Package of 12 Strips

Intensiv ProxoStrip Plus (grit size 40µm and 15µm)
Ref. PXP4015/6 Package of 6 Strips
Ref. PXP4015/12 Package of 12 Strips

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